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Intended use

The LDL-C Calibrator is designed to be used with LDL-C L-Type reagent for the quantitative determination of Low Density Lipoprotein Cholesterol (LDL-C) in serum.

Reagents and ingredients

LDL-C CAL: 3 bottles x for 1 mL
Contains human serum and stabilizer.

Reagent preparation

Accurately add 1 mL of distilled or deionized water to dissolve the contents of each of the Calibrator bottles. The reconstituted calibrator is stable for 7 days at 2-10°C.

Assigned values and their determination

Assigned values: see bottle labels.
Determination: Based on the US CDC LDL-C Reference method.

Storage condition

Store at 2-10°C.
Expiration date: see bottle labels or package.

Safety instructions:

Warning. May Cause allergic skin reaction.
Avoid breathing dust/fume/gas/mist/vapours/spray. IF ON SKIN (or hair): Remove/ Take off immediately all contaminated clothing. Rinse skin with water/shower. If skin irritation or rash occurs: Get medical advice/attention.

Hazard-determining components:

Contains as preservative:
5-chloro-2-methyl-2H-isothiazol-3-one [EC No. 247-500-7] and
2-methyl-2H-isothiazol-3-one [EC No. 220-239-6] (3:1)

Warnings and precautions

- For *in vitro* diagnostic use.
- Only for professional use.
- Not to be used internally in humans or animals.
- Do not use the calibrator after the date of expiry indicated on the packaging
- Do not use the calibrator for any other purpose than described herein.
- Human serum material was used in the manufacture of this product. The raw human serum used was tested and found negative for HBsAg, anti-HIV antibody and anti-HCV antibody. Because no test can offer complete assurance that products derived from human blood will not transmit infectious agents, it is recommended that this product should be handled with the same biohazard precautions used for patient specimens.
- When discarding the product, do so in accordance with local and national regulation

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