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Universal calibrator for use in tests for quantitative in vitro determination of various analytes on photometric systems.

#### Description

DunaCal is a lyophilized calibrator (human serum based).

#### Storage

The unopened bottles of DunaCal must be stored at 2-8°C.

#### Stability

Unopened until the end of the indicated month of expiry. Once reconstituted DunaCal can be used within the period reported in the table below if stored tightly closed at the indicated temperature.

+ 4 °C	
Bilirubin (in the dark)	8 hours
Other analytes	2 days
Acid Phosphatase (Total and Prostatic)	2 days
+25°C	
Bilirubin (in the dark)	4 hours
Other analytes	8 hours
Acid Phosphatase (Total and Prostatic)	2 hours
-20°C*	
Bilirubin	14 days
Other analytes	30 days
Acid Phosphatase (Total and Prostatic)	30 days

\* freeze only once!

#### Warnings and Precautions

Each individual blood donation used for production of DunaCal was found to be non-reactive when tested with approved methods for HBsAg, anti-HIV 1+2 and anti-HCV. As there is no possibility to exclude definitely that products derived from human blood transmit infectious agents, it is recommended to handle the calibrator with the same precautions used for patient specimens.

#### Preparation

The lyophilisate is vacuum sealed, therefore the vial should be opened very carefully to avoid loss of dried material. For reconstitution add with a pipette exactly 3 ml of distilled water. Close the vial carefully and allow the calibrator to stand for 30 minutes swirling occasionally. Avoid foaming! Do not shake!

When measuring alkaline phosphatase the calibrator must be allowed to stand for 2 hours at + 25 °C.

#### Procedure

Please refer to the reagent package insert for instructions for use.

#### For in vitro diagnostic use only.

#### Literature

- Dati F. Reference materials and guidelines for standardization of methods in laboratory medicine. In: Thomas L, editor. Clinical laboratory diagnostics. 1<sup>st</sup> ed. Frankfurt: TH-Books Verlagsgesellschaft; 1998. p. 1404-26.
- Moss DW, Henderson AR. Enzymes. In: Burtis CA, Ashwood ER, editors. Tietz Textbook of Clinical Chemistry. 2<sup>nd</sup> ed. Philadelphia: W.B Saunders Company; 1994. p. 735-896.
- Biosafety in Microbiological and Biomedical Laboratories, U.S. Department of Health and Human Services, Washington 1993 (HHS Publication No. [CDC] 93-8395)

Manufacturer: Diagnosticum Ltd. Attila u. 126., H-1047 Budapest, Hungary

The following symbols can be used on the labels

	In vitro diagnostic device		Batch code
	Manufacturer		Catalogue number
	CE-marking		This way up
	Temperature limitations		Biological risk
	Use by (year/month)		Contents of the kit
	Calibrator		

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